ProClear™
Aligner Custom Guide

INSIDE YOUR ALIGNER KIT:

1

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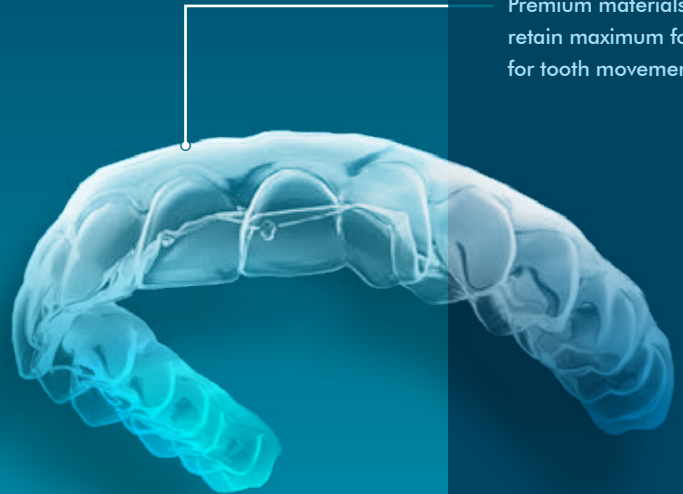
Keep 'Em Nice & Clean!

PROPER CARE

- Wear your aligners daily for best results. You may remove them for eating, drinking, brushing your teeth, flossing and cleaning.
- Always rinse your aligners when you remove them. We recommend you brush and floss your teeth before reinserting your aligners.
- To clean your aligners, gently brush them with a toothbrush and rinse them in luke warm water. Clear antibacterial soft soap works best for cleaning. Using anything but clear soap can cause your aligners to take on alternative hues and shades. Never use hot water on your aligners - it could permanently warp the plastic.

Premium materials retain maximum force for tooth movement.

Soaking them in a clear retainer or denture cleaner helps keep them clean, clear and sanitized.

Whiten & Brighten That Smile

The best time to whiten your teeth is just before bed, after you've brushed your teeth. Wear whitening gel for 30 minutes daily.

Included as a part of your ProClear™ Aligner Kit is our powerful whitening kit. The ProClear™ Teeth Whitening Kit is the perfect, professional way to whiten your teeth at home.

Our kit uses LED light technology along with our whitening gel for faster, longer-lasting results. It illuminates a brilliant blue light and rapidly removes surface stains, while also penetrating deep to lift out embedded stains.

FEATURES:

- Applies in seconds
- Safe and easy to use
- Removes years of stains
- Delivers professional results at home

A close-up photograph of a man's lower face, focusing on his mouth as he smiles. His teeth are bright white and appear healthy. He has a light beard and mustache. The background is plain white.

FOR BEST RESULTS:

Repeat daily until ideal shade of white is achieved. Track your progress using our Whitening Shade Guide.

FREQUENTLY ASKED QUESTIONS

I feel soreness when I wear my aligners is this normal?

Yes. Pressure and soreness with a new set of aligner trays is completely normal. The aligners are moving your teeth. Leave them in for as long as possible. For discomfort, you can take a pain reliever until your teeth move to accommodate the new tray and the soreness subsides.

The trays are irritating my gums and cheeks?

It is normal for the plastic edges of the aligners to poke out. This may cause irritation or cut your gums or inner cheeks. Should your aligners irritate your gums and cheeks, simply use the emery board to gently file, buff and smooth only the rough, protruding edges or areas that cause discomfort.

I lost one of my trays what do I do?

If you lose or damage your aligner, simply put in the last one you wore in your series to repeat the previous step and contact us 1-800-324-0750 to ship you a replacement. One of our customer service representatives will guide you through the next stage and make sure you are back on track to improving your smile.

My aligners are foggy and my breath is starting to smell. What should I do?

We strongly recommend that you keep your aligners clean and always brush and floss after every meal. Do not eat or drink while wearing your aligners (only cool water). Use a clear antibacterial soap in luke warm water to clean. You may also soak them in a clear retainer or denture cleaner.

When and how long do I wear my ProClear™ Aligners?

Everyday. Wear your ProClear™ Aligners daily as outlined in your custom plan. They may be removed for eating, drinking, brushing your teeth and cleaning.

What type of results should I expect after completing my ProClear™ Custom Plan?

Best results are achieved when you follow your custom guide including the length of time the aligners are worn daily.

NEED ADDITIONAL HELP?

Visit us online for additional help,
including tips and tutorials at

www.ProClearAligners.com/Support

SHOP KITS & ACCESSORIES

Order your ProClear™ Aligner Kit and
whitening gel refills online at

www.ProClearAligners.com/Shop

The Professional Way To Get a Beautiful Smile
Without Going To The Dentist

www.ProClearAligners.com